

Rely+On™ Virkon™

Hospital Grade Surface Disinfectant

Tablets / Powder

Not intended to be used on therapeutic devices

AUST L 94340

Manufactured by:

Antec International Limited
Windham Road, Chilton Industrial Estate
Sudbury, Suffolk, CO10 2XD, United Kingdom
Tel: +44 (0) 1787 377 305
Fax: +44 (0) 1787 310 846
relyondisinfection@lanxess.com
www.relyondisinfection.com

**IXOM Emergency Response Service (ERS) Phone:
1 800 033 111 (24 hrs) toll free Australia wide**

Distributed by: Med-Con Pty. Ltd.

P.O. Box, 6159, Shepparton, Victoria,
Australia 3632
Toll Free: 1800 001 435
sales@medcon.com.au

57784885/Virk/IFU/Medcon,Australia/22.05.20/Z

SURFACES

Hard Surfaces: Floors, laboratory benches, work tops, bed frames, tables, sinks, baths, toilets, etc. in Healthcare Institutions, Laboratories, Food and Catering establishments and all public and private environments.

1. Prepare a 1% solution as per directions on label or overleaf.
2. Small areas: apply Rely+On™ Virkon™ solution via a spray bottle. Large areas: apply Rely+On™ Virkon™ solution with a cloth or mop for floors.
3. Leave solution on the surface for 10 minutes, wipe dry with absorbent paper towel or cloth to remove any residue and then discard to waste.

Soft Surfaces and Fabrics: Carpets, floor coverings, mattresses, curtains and upholstery. As with all cleaners/ disinfectants, colour-fastness of fabrics and textiles should be established before application.

1. Prepare a 1% solution as directed.
2. Apply Rely+On™ Virkon™ solution from a spray bottle.
3. Allow 10 minutes contact before removing excess solution with an absorbent cloth or paper.
4. Allow to dry and remove any residues by vacuum cleaner.

LABORATORY EQUIPMENT

(Centrifuges, Clinical Analysers, pipettes etc.)

1. Prepare 1% solution as directed.
2. a) Equipment: wipe over surfaces which may be contaminated with infected material. Sample probes and tubing can be decontaminated by aspiration of Rely+On™ Virkon™ solution. Always check compatibility of with manufacturer of equipment.
b) Discard jars: fill jar or container with fresh solution daily.

BODY FLUID SPILLAGES (blood, urine, vomitus, etc)

Hard Surfaces: Dispense Rely+On™ Virkon™ powder or crush a tablet directly onto the spillage until all liquid is absorbed. Scrape or sweep damp powder into container for disposal. After removal of damp powder, apply a 1% solution to the affected area and items used for disposal. Allow 10 minutes contact and wipe to remove solution/ residue with cloth or paper towels.

Soft furnishings and carpets: Before use, test for colour-fastness of material with powder and solution, then use as described for hard surfaces.

LAUNDRY

Soak soiled bed-linen or table cloths etc. for 10 minutes in 1% Rely+On™ Virkon™ solution before laundering. Check colour-fastness before use.

Add 5g of powder or 1x5g tablet per 500mls of lukewarm water. Mix thoroughly until completely dissolved to form a pink coloured solution.

| Powder | Tablet | Water |
|--------|--------|----------|
| 5gm | 1 | 500mls |
| 10gm | 2 | 1 Litre |
| 20gm | 4 | 2 Litres |
| 50gm | 10 | 5 Litres |

1% Rely+On™ Virkon™ solutions are stable for up to 5 days. Recommended good practice is that solutions are replenished and discarded daily. Rely+On™ Virkon™ solution must be discarded when the pink colour fades or after 5 days, whichever is sooner. Always rinse with potable water.

STORAGE:

Store tightly closed containers in a cool, dry place. Store prepared solution out of direct sunlight at ambient temperature (15-25°C).

PRECAUTIONS:

As with all disinfectants, wash splashes of solution from skin and eyes immediately. Do not mix with detergents or other chemicals. DO NOT prepare concentrated solution for later dilution.

SDS: Additional information is listed in the Safety Data Sheet.